Instructions For Use

Futar[®]

Vinyl polysiloxane bite registration
material DIN 13903

Made in Germany
37273/1119

Gebrauchsinformation
Instructions For Use
Mode d'emploi
Gebruiksaanwijzing
Информация по использованию
Istruzioni per l'uso
Instrucciones de uso
Kullanım Kılavuzu
Instruções de uso
Instrukcja użycia
사용 지침
取扱説明書
تعليمات الاستخدام

**Inhalt/Content/Contenu/Inhoud/содержимое/Contenuto/Contenido/
İçerik/Conteúdo/Treść/목차/目次/الفهرس**

Deutsch	4	日本語	52
English	8	عربي	57
Français	12		
Nederlands	17		
Русский	21		
Italiano	26		
Español	30		
Türkçe	34		
Português	38		
Polski	43		
한국어	48		

Vinyl polysiloxane bite registration material DIN 13903

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Product Description

Futar[®], Futar[®] Fast, Futar[®] D, Futar[®] D Fast, Futar[®] D Slow and Futar[®] Cut & Trim Fast are syringeable addition-curing elastomeric bite registration materials intended for making accurate occlusal records.

Indications

- All Futar[®] bite registration materials (Futar[®], Futar[®] Fast, Futar[®] D, Futar[®] D Fast, Futar[®] D Slow, Futar[®] Cut & Trim Fast) can be used for making accurate occlusal registrations.
- Futar[®] D materials (Futar[®] D, Futar[®] D Fast, Futar[®] D Slow, Futar[®] Cut & Trim Fast) can be used for applications, which require a hard bite registration silicone.
- Futar[®] Fast, Futar[®] D Fast and Futar[®] Cut & Trim Fast can be used for “small applications” because of their shorter working time.
- Futar[®] D Slow can be used for time-consuming bite registrations because of its longer working time.

- Futar[®] Cut & Trim Fast can also be used for an optical registration of occlusal data for CAD/CAM systems without using powder.

Warnings

Do not use Futar[®] bite registration materials for conventional impressions as their high end hardness values severely inhibit withdrawal from the mouth and may result in inadvertent extraction of teeth.

Cautions

Do not use after expiration date.

Do not leave any residual bite registration material in the sulcus or oral cavity.

Do not swallow bite registration material! If swallowed: consult a medical doctor if problems arise or persist.

Avoid contact with the eyes. If there is accidental contact with the eyes: rinse immediately and thoroughly with an eye wash or water. Consult a medical doctor if problems arise or persist.

The product does not normally cause allergic reactions; however, for sensitive persons, an allergic reaction to the product cannot be ruled out.

Use of products containing active sulfur, aluminum chloride or nitrogen compounds (retraction cords containing ferric sulfate, polysulfide impression materials etc.) in conjunction with this product will interfere with the setting reaction of the vinyl polysiloxane material. Use of these materials requires the area to be rinsed thoroughly to remove all residues.

If gloves are worn when handling the material, they should be tested with a sample of the bite registration material beforehand to ensure compatibility. Do not use latex gloves.

Do not allow the material to enter the sewer or water system to avoid environmental hazard.

Avoid contact with clothing, as the material cannot be removed by dry cleaning.

Note

The total working time and intraoral setting time should be strictly adhered to in order to ensure optimal bite registrations. Should the temperature of the material deviate noticeably from 23 °C (74 °F), the total working time and setting times will be affected.

Instructions for use

Insert the cartridge into the Applyfix® 4 dispensing gun DS-50 1:1/2:1. Ensure that the notches on the cartridge base are pointing downwards. The clasp will not close if the cartridge

is not properly inserted into the dispensing gun.

Remove the cartridge cap. The cap can be replaced after initial use.

Advance plunger into cartridge and dispense a small amount of impression material until equal amounts are extruded at the same rate.

Install a mixing tip on the cartridge and turn cap 90 degrees in a clockwise direction to lock in place.

Inject material directly onto the patient's occlusal surfaces or occlusal rims.

Replace the plug after use. Before using the cartridge again, remove the cartridge cap or the filled mixing tip and check the cartridge outlet openings to ensure that no polymerized materials have sealed the openings. Clear any clogged openings and dispense a small amount of impression material until equal amounts are extruded at the same rate.

Install a new mixing tip and continue as described above.

Cleaning and disinfection of the Applyfix® 4:

- Remove the cartridge.
- Remove any uncured paste using a cloth soaked in alcohol.
- The Applyfix® 4 dispensing gun can be used several times if properly disinfected. Kettenbach recommends

exchanging the gun if there are visible traces of use. Disinfection can be carried out using a commercially available dipping disinfectant. Kettenbach recommends the use of glutaraldehyde-based solutions.

Storage Conditions

Store in a dry place at room temperature; do not expose to sunlight and avoid exposure to extreme temperature fluctuations.

Disinfection

Bite registrations can be disinfected using for example, a 2% acidic glutaraldehyde solution. Use disinfection solutions. See manufacturer instructions.

Trademarks

Applyfix® 4 and Futar® are registered trademarks of Kettenbach GmbH & Co. KG.

Note

Selected Kettenbach materials/products are available in certain markets only.

For professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

Limitation of liability

Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development.

However, users must comply with and consider all recommendations and information in connection with any use.

Updated: March 15th, 2019

Product Overview and Technical Data

Product name	Working time at 23° C / 74° F ≤	Intraoral setting time at 35° C / 95° F ≥	Total setting time* ≥	Hardness (approx.) Shore	Linear dimensional change (approx.) %	Mixing technique	Mixing element
Futar®	30 seconds	1 minute 30 seconds	2 minutes	A 90/HD 58	- 0.20	Applyfix® 4 dispensing gun DS-50 1:1/2:1	Mixing tip, green, Ø 6.5 mm
Futar® Fast	15 seconds	45 seconds	1 minute	A 90/HD 58	- 0.20	Applyfix® 4 dispensing gun DS-50 1:1/2:1	Mixing tip, green, Ø 6.5 mm
Futar® D	30 seconds	1 minute 30 seconds	2 minutes	D 43/HD 76	- 0.10	Applyfix® 4 dispensing gun DS-50 1:1/2:1	Mixing tip, green, Ø 6.5 mm
Futar® D Fast	15 seconds	45 seconds	1 minute	D 43/HD 76	- 0.10	Applyfix® 4 dispensing gun DS-50 1:1/2:1	Mixing tip, green, Ø 6.5 mm
Futar® D Slow	1 minute 30 seconds	3 minutes	4 minutes 30 seconds	D 43/HD 76	- 0.10	Applyfix® 4 dispensing gun DS-50 1:1/2:1	Mixing tip, green, Ø 6.5 mm
Futar® Cut & Trim Fast	15 seconds	45 seconds	1 minute	D 35/HD 60	- 0.20	Applyfix® 4 dispensing gun DS-50 1:1/2:1	Mixing tip, yellow, Ø 4.2 mm

*Total setting time (removal time from mouth) from start of mix.

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