



 **SUPER WITH FUTAR®**

BITE REGISTRATION



KETTENBACHDENTAL
Simply intelligent

FUTAR® – SIMPLY IMPRESSIVE!

Bite registration

The **Futar®** product family includes eight different bite registration materials based on A-silicone for creating accurate occlusal records.

SUPER IN QUALITY

- For accurate occlusal registrations

SUPER THROUGH VERSATILITY

- Choose between high and highest final hardness, in addition to different processing times!

SUPER IN RELIABILITY AND CONTINUITY

- Tried and tested many times with always reliable quality

Do not make compromises with your bite registration!



FUTAR® – THE ADVANTAGES AT A GLANCE

Futar® / Futar® Fast:

- High final hardness (Shore-A 90)
- Flexible in processing: easy to cut
- Also available with fast-setting characteristics for more patient comfort!

Futar® Easy / Futar® Easy Fast:

- High final hardness (Shore-A 90)
- Low bite resistance
- Flexible in processing: easy to cut
- Also available with fast-setting characteristics
- Low dispensing force – easy to apply



The Futar® family can
do even more:

Futar® D / Futar® D Fast:

- Extra high final hardness (Shore-D 43)
- Offers very good millability
- Also available with fast-setting characteristics: for more patient comfort!



- Accurate results thanks to excellent flow properties
- Low elasticity reduces springiness in the model assignment
- Surplus can be broken off easily

Futar® Clear Fast:

- No risk of fracture caused by comfortable final hardness (Shore-A 62)
- Enables direct visual control of the correct bite













Futar® D Slow:

- Extra high final hardness (Shore-D 43)
- Extra long processing time for time-consuming applications, e.g. for myocentric bite registration, fixation of impression posts or individual contouring of the margins
- Versatile use also in the laboratory

**SIMPLE,
CLEAN AND
FAST TO USE!**











FUTAR® – PERFORMANCE RUNS IN THE FAMILY

Dental arch

 Bite registration material	 Mixing	 Working time at 23 °C/74 °F ≤	 Intraoral setting time at 35 °C/95 °F ≥	 Total setting time ²⁾ ≥	 Hardness (approx.) Shore
Futar®		30 sec.	90 sec.	2:00 min.	A 90
Futar® Easy		30 sec.	90 sec.	2:00 min.	A 90
Futar® D		30 sec.	90 sec.	2:00 min.	D 43
Futar® D Slow		90 sec.	3:00 min.	4:30 min.	D 43

²⁾Total setting time (removal time from mouth) from the start of mixing

Segment

 Bite registration material	 Mixing	 Working time at 23 °C/74 °F ≤	 Intraoral setting time at 35 °C/95 °F ≥	 Total setting time ²⁾ ≥	 Hardness (approx.) Shore
Futar® Fast		15 sec.	45 sec.	1:00 min.	A 90
Futar® Easy Fast		15 sec.	45 sec.	1:00 min.	A 90
Futar® D Fast		15 sec.	45 sec.	1:00 min.	D 43
Futar® Clear Fast		15 sec.	45 sec.	1:00 min.	A 62

²⁾Total setting time (removal time from mouth) from the start of mixing

**CREATE
YOUR BITE
REGISTRATION
IN JUST ONE
MINUTE!**

FUTAR® – THE RANGE



Products	Normal pack 2 x 50 mL cartridges, 6 blue mixing tips
Futar®	REF 1191211
Futar® Fast	REF 1192611
Futar® Easy	REF 1192311
Futar® Easy Fast	REF 1192411
Futar® D	REF 1193211
Futar® D Fast	REF 1196111
Futar® D Slow	REF 1195111
Futar® Clear Fast	REF 1192811



NOTES

Kettenbach LP
 16052 Beach Blvd. Ste. 221
 Huntington Beach, CA 92647, USA
 Phone: 877-532-2123
 Fax: 888-892-9820
 Website: www.kettenbach-dental.us
 Web shop: www.kettenbachusa.com
 Custservice@kettenbachusa.com

Not for use, sale and
 distribution in Germany.



KETTENBACHDENTAL
 Simply intelligent