Vinylsiloxanether® impression material
DIN EN ISO 4823



IDENTIUM®

INSTRUCTIONS FOR USE / Gebrauchsinformation / Mode d'emploi / Gebruiksaanwijzing /
Инструкция по использованию / Istruzioni per l'uso / Instrucciones de uso / Kullanım Kılavuzu /
Instruções de uso / Instrukcja użycia / Інструкція для застосування / 取扱説明書 / تعليمات الاستخدام

CONTENT

Inhalt / Contenu / Inhoud / содержимое /
Contenuto / Contenido / İçerik / Conteúdo /
Treść / Зміст / 目次 / الفهرس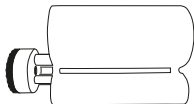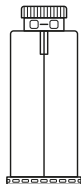

Deutsch	4
English	14
Français	23
Nederlands	33
Русский	43
Italiano	54
Español	64
Türkçe	74
Português	84
Polski	94
Українська	105
日本語	117
127	عربي

Vinylsiloxanether® impression material
DIN EN ISO 4823

Identium® Heavy

Identium® Heavy Fast

Identium® Medium

Identium® Medium Fast

Identium® Light

Identium® Light Fast

For dental professional use only.

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Product Description

Identium® products are addition-curing, elastomeric impression materials with parts of vinyl polysiloxane and polyether (Vinylsiloxanether®). They convince with excellent flow and hydrophilic properties, high tear strength, dimensional accuracy, and resistance to permanent deformation.

The Identium® product range consists of three different viscosities (heavy bodied, medium bodied, light bodied), available in two delivery systems: standard 1:1 (50 ml auto-mix cartridges) and 5:1 (380 ml cartridges) for use in most commercially available other automatic dispensing and mixing machines, e.g., Sympress.

The Identium® products are all available in a regular and fast version.

Indications / Techniques

Identium® Heavy/Identium® Heavy Fast is to be used as a heavy bodied material in one-step technique (double mix) for:

- Impressions for crowns/bridges, inlays/onlays and veneer preparations
- Functional impressions

- Impressions for full or partial dentures
- Implant impressions

Identium® Medium /Identium® Medium Fast is to be used as a medium bodied tray or syringeable impression material in one-step technique (monophase or double mix) for:

- Impressions for crowns/bridges, inlays/onlays and veneer preparations
- Implant impressions
- Fixation impressions
- Functional impressions

Identium® Light/Identium® Light Fast is to be used as syringeable impression material in one-step technique (double mix) for:

- Impressions for crowns/bridges, inlays/onlays and veneer preparations
- Reline impressions
- Impressions for full or partial dentures

Patient Target Group

Patients being treated in the context of a dental procedure.

Warnings

Do not use Identium® impression materials for two-step putty-wash impression technique or temporary reliner.

Do not use Identium® impression materials with other elastomeric impression materials.

Potential Allergenic Substances

Platinum compound, 4-methoxyphenol.

Cautions

Do not use after expiration date.

Normally, impression materials do not cause allergic reactions; however, an allergic reaction cannot be ruled out in sensitive individuals.

If in doubt, we recommend performing an allergy test before use.

Do not use the material in case of known allergies to any of the ingredients or in case of contact allergies.

Do not leave any residual impression material in the sulcus or oral cavity.

Do not swallow impression material! If swallowed: Consult a medical doctor if problems arise or persist.

Avoid contact with the eyes. If there is accidental contact with the eyes: Rinse immediately and thoroughly with an eye wash or water. Consult a medical doctor if problems arise or persist.

Use of products containing active sulfur, aluminium chloride or nitrogen compounds (retraction cords containing ferric sulfate, polysulfide impression materials, etc.) in conjunction with this product will interfere with the setting reaction of the Vinylsiloxanether® material. Use of these materials requires the area to be rinsed thoroughly to remove all residue. Avoid contact with latex gloves.

When taking impressions of areas with severe undercuts and wide interdental spaces, standard blocking-out techniques must be used.

When using a custom impression tray, ensure that sufficient space remains between the side of the tray and the teeth/jaw.

Do not allow the material to enter the sewer or water system to avoid environmental hazard.

Avoid contact with clothing, as the material cannot be removed by dry cleaning.

Note

To ensure optimal impressions, the temperature of the material should not deviate from 23°C (74°F) before applying. Otherwise, working and setting times will be affected.

Impression tray: Preparation and Adhesives

In principle, all common impression trays can be used if a relevant dynamic pressure is guaranteed. When retention is not sufficient, to ensure a strong bond to the impression material, brush the impression tray with a thin film of Identium® Adhesive prior to loading the tray with impression material. Allow to dry as per instructions for use and cover with a sufficient amount of impression material.

Instructions for the Safe Handling of the Kettenbach 380-mL cartridge system; 5:1

Components: 380-mL cartridge, Dynamic Mixers

Prior to first use please unscrew the blue plug counter-clockwise and pull it out (Figure 1, Page 136). Check the outlet opening. Then screw on the dynamic mixer (Figure 2, Page 136). Ensure that the positioning aid is on the top.

Please use only the dispensing and mixing components compatible with the Kettenbach 380 ml cartridge system for dispensing and mixing impression material.

The Kettenbach 380 ml cartridge is heavy and could be damaged if dropped. Damaged cartridges must not be used under any circumstances.

When using the material in a Sympress II dispensing device, please activate the speed setting indicated on the product label (Figure 6, Page 137). Failure to comply with this instruction

may modify the curing characteristics of the impression materials.

If you are using a unit other than the Sympress mixing device, please follow the instructions for the automatic dispensing and mixing system you are using.

When changing to a different mixing device, always discard at least 3 cm of mixed material.

Using the Kettenbach 380-mL cartridge system; 5:1 in Dispensing Device Sympress / Sympress II

By pressing the plunger return knob, allow the plunger to return to its original position automatically (while the cover is closed; Figure 3, Page 136). Next, open the cover (Figure 4, Page 136).

Insert the Kettenbach 380 ml cartridge with the pre-assembled mixer in the Sympress/Sympress II dispensing and mixing unit (Figure 5, Page 137). If you are working with a previously used cartridge, the filled mixer will already be located on the cartridge as a cap.

If you are using a Sympress II dispensing device, please activate the speed setting indicated on the product label. A light signal will confirm your selection (Figure 6, Page 137).

Close the device cover (Figure 7, Page 137).

Using the Cartridge for the First Time

Dispense the impression material (ensure that the dynamic mixer is wound into place fully) until the base material and catalyst are mixed homogeneously; discard at least the first 3 cm when dispensing (Figure 8, Page 137).

At least 3 cm of the mixed material must be discarded each time the plunger is restarted or when using the cartridge in different mixing units.

Load the impression tray (or application syringe) with the required amount of material (Figure 9, Page 137) by pressing the appropriate feed key. Hold the impression at an angle when loading and press against it lightly. Leave the mixer in the material when dispensing to avoid air bubbles.

Release the feed key as soon as the required amount of material has been dispensed. The filled dynamic mixer should remain on the cartridge as a seal.

Remove the used mixer before the next application by turning it to the left (Figure 10, Page 137), visually check the outlet opening for blockage and attach a new dynamic mixer (Figure 10, Page 137).

Remove the completely empty jumbo cartridge from the dispensing and mixing unit and dispose of it according to regional regulations.

Further Use of a Cartridge Already in Use

Wind a new dynamic mixer onto the same mixing unit and begin the new application (material no longer has to be discarded).

Instructions on the Use of Cartridge Material in the Applyfix 4 Dispensing Gun DS-50 1:1/2:1

Insert the cartridge into the Applyfix 4 dispensing gun DS-50 1:1/2:1.

Ensure that the notches on the cartridge base are pointing downwards. The clasp will not close if the cartridge is not properly inserted into the dispensing gun.

The cartridge's tamper-evident closure can now be turned (see Fig. 1).

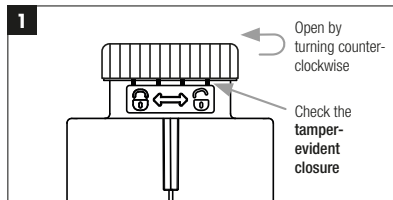
Abb. 1: Opening the tamper-evident closure

The tamper-evident closure ensures that this is an intact original product from Kettenbach Dental.

When used for the first time, the cartridge closure is visibly and audibly separated from the cartridge body. After use, the closure cap can be used to close the cartridge.

Insert the mixing tip with the integrated positioning aid into the coding hole provided for that purpose and rotate clockwise into the thread (see Fig. 2).

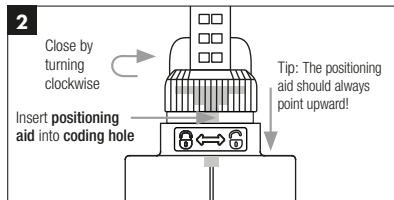
Abb. 2: Inserting the mixing tip

The intuitive handling significantly reduces mistakes when inserting.

Push the plunger into the cartridge and begin applying immediately.

The mixing system ensures even delivery at all times, so it is no longer necessary to throw out the impression material.

Either dispense the necessary amount of impression material directly into the impression tray, on to the preliminary impression or into an impression syringe. Use the cartridge with the mixer attached and intraoral tip or the impression syringe to apply the material around the prepared teeth.

Replace the closure cap after use. Before using the cartridge again remove the closure cap, attach a new mixing tip and proceed as described above.

Cleaning and Disinfection of the Applyfix 4

Remove the cartridge.

Remove any uncured paste using a cloth soaked in alcohol.

The Applyfix 4 dispensing gun can be used several times if properly disinfected. Kettenbach recommends exchanging the gun if there are visible traces of use. Disinfection can be carried out using a commercially available dipping disinfectant. Kettenbach recommends the use of glutaraldehyde-based solutions.

Instructions for Use: Applyfix 5 Application Syringe

Insert a new syringe tip into tip retainer. Screw tip retainer with the syringe tip onto the syringe cylinder (Figure 2, Page 138).

If necessary, lubricate the ring washer very slightly with petrolatum for an easy action of the syringe piston.

Prepare the preferred impression material (the Applyfix 5 application syringe is indicated for light bodied or medium bodied consistency impression material only).

Attach the open syringe cylinder of the Applyfix 5 directly to the mixing tip of the preferred impression material (Figure 3, Page 138).

Fill the application syringe with the required amount of impression material by pressing the *feed button* (Fig. 9, page 137). Insert the syringe piston into the syringe cylinder (Figure 4, Page 138) and apply the impression material into the patient's mouth as desired.

Cleaning and Disinfection of the Applyfix 5

Clean the Applyfix 5 application syringe with the brush (Figure 5, Page 138) after using and removing of the full hardened impression material. The Applyfix 5 application syringe can be used several times if properly disinfected. Kettenbach recommends exchanging the gun if there are visible traces of use. Disinfection can be carried out using

a commercially available dipping disinfectant. Kettenbach recommends the use of glutaraldehyde-based solutions. The Applyfix 5 can be reused and autoclaved several times and should be replaced once it shows signs of mechanical damage. Do not reuse the syringe tips, to avoid contamination.

Cautions

Only use one material per Applyfix 5 application syringe. Contamination of different impression materials renders the impression material unusable.

Storage Conditions

Store in a dry place at room temperature; do not expose to sunlight and avoid exposure to extreme temperature fluctuations.

Disinfection

Impressions can be disinfected using for example a 2% acidic glutaraldehyde solution. Use disinfection solutions specific for impression materials. See manufacturer instructions.

Model Preparation

Impressions with Identium® may be poured immediately after disinfection of the impression up to several weeks with high quality dental stone (class III–V). Impressions can also be poured several times.

Composition

Vinyl polysiloxanes, Hydrogen polysiloxanes, Filler, Pigments, Additives, Platinum compound.

Disposal

The empty cartridges/mixing tips/applicator tips and set material can be disposed in the household rubbish.

Trademarks

Identium® is a registered trademark of Kettenbach GmbH & Co. KG.

Note

Selected Kettenbach materials are available in certain markets only.

For dental professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

If serious incidents occur that are related to the product, please immediately contact the manufacturer and the competent authority.

Limitation of Liability

Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and information in connection with any use.

Updated: July 19, 2023

Product Overview

Product name	ISO 4823	Consistency (approx.) mm	Mixing ratio and total content volume per unit	Mixing technique	Mixing element
Identium® Heavy	Type 1, Heavy bodied	32	5:1, 380 ml cartridge	Sympress/Sympress II or other automatic dispensing and mixing unit	Dynamic mixer, blue
Identium® Heavy Fast	Type 1, Heavy bodied	32	5:1, 380 ml cartridge	Sympress/Sympress II or other automatic dispensing and mixing unit	Dynamic mixer, blue
Identium® Medium	Type 2, Medium bodied	36	5:1, 380 ml cartridge	Sympress/Sympress II or other automatic dispensing and mixing unit	Dynamic mixer, blue
Identium® Medium Fast	Type 2, Medium bodied	36	5:1, 380 ml cartridge	Sympress/Sympress II or other automatic dispensing and mixing unit	Dynamic mixer, blue
Identium® Light	Type 3, Light bodied	42	1:1, 50 ml cartridge	Applyfix 4 dispensing gun DS-50 1:1 / 2:1	Mixing tip, red, Ø 4.0 mm
Identium® Light Fast	Type 3, Light bodied	42	1:1, 50 ml cartridge	Applyfix 4 dispensing gun DS-50 1:1 / 2:1	Mixing tip, red, Ø 4.0 mm

Technical Data

Product name	Mixing ratio	Working time at 23°C/74°F ≤	Working time at 35°C/95°F ≤	Intraoral setting time at 35°C/95°F ≥	Total setting time* ≥	Hardness (approx.) Shore	Linear dimensional change (approx.) %	Elastic Recovery test (approx.) %	Strain in compression (approx.) %
Identium® Heavy	5:1	2 minutes	not applicable	2 minutes 30 seconds	4 minutes. 30 seconds	A 60	– 0.20	99.0	2.3
Identium® Heavy Fast	5:1	1 minute 15 seconds	not applicable	2 minutes 15 seconds	3 minutes 30 seconds	A 60	– 0.20	99.0	2.3
Identium® Medium	5:1	2 minutes	1 minute 20 seconds	2 minutes 30 seconds	4 minutes 30 seconds	A 60	– 0.20	99.0	2.3
Identium® Medium Fast	5:1	1 minute 15 seconds	40 seconds	2 minutes 15 seconds	3 minutes 30 seconds	A 60	– 0.20	99.0	2.3
Identium® Light	1:1	2 minutes	1 minute 20 seconds	2 minutes 30 seconds	4 minutes 30 seconds	A 50	– 0.20	99.0	3.8
Identium® Light Fast	1:1	1 minute 15 seconds	40 seconds	2 minutes 15 seconds	3 minutes 30 seconds	A 50	– 0.20	99.0	3.8

* Total setting time (removal time from mouth) from start of mix

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