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English

Product Description

Profisil® Fluoride Varnish Plus is designed to securely adhere to the tooth surface for several hours while releasing fluoride ions. The varnish contains 5 % sodium fluoride suspended in a mucosa-friendly, pleasantly flavored dimethicone gel. The varnish is available in following versions: mint, berry and unflavored.

Indications

Profisil® Fluoride Varnish Plus is a 5 % sodium fluoride varnish which produces mechanical occlusion of the dentinal tubules in the treatment of tooth hypersensitivity.

Patient Target Group

Patients of all genders and ages including paediatric, adolescent, adult, and elderly individuals.

Potential Allergenic Substances

Natural and artificial berry flavor.

Contraindications

Ulcerative gingivitis and stomatitis.

Do not use this product on any individual who has a known intolerance to fluoride. If any signs of inflammation or allergic reaction occur, discontinue use immediately and refrain from further use with this patient.

Warnings

Allergic reactions to the product cannot be ruled out in sensitive persons.

Sensitization through contact with skin / mucous tissue is possible.

Cautions

Keep out of reach for children.

Do not use after the expiration date.

Do not use other products containing fluoride within 24 hours of using the varnish. Take the appropriate protective measures (e.g. wear gloves and safety glasses).

Avoid contact with the eyes. If there is accidental contact with the eyes: rinse immediately and thoroughly with an eye wash or water. Consult a medical doctor if problems arise or persist.

Avoid contact with clothing, as permanent damage to clothes cannot be excluded.

For hygienic reasons, the components are intended for single use only.

Safety data sheets are available at www.kettenbach.com.

General Notes

Profisil® Fluoride Varnish Plus can be applied to all tooth surfaces, even to difficult accessible areas. A prophylactic cleaning is not required prior to use. If a prophylactic

cleaning is performed, apply the varnish following scaling and polishing. Unnecessary to dry teeth prior to application of the varnish gel.

Instruction for use

Prior to use, visually inspect the single dose blister for damage and do not use it if damaged or already opened.

Peel off the single dose blister foil carefully. Lift the foil near the brush and remove the brush. Peel back the remaining foil to expose the varnish.

Using the supplied applicator brush, mix contents briefly to homogenize the contents prior to application using the non-brush end of the applicator brush. Constituents of varnishes tend to settle on the bottom of the package in storage.

Using the supplied applicator brush, apply a thin, even layer to all affected surfaces of the dentition.

The single dose blister contains sufficient varnish to cover all affected teeth of both the upper and lower arch. Do not rinse or suction immediately after use.

After application of the varnish, instruct the patient to refrain from brushing and flossing, avoid hot drinks and products containing alcohol (i.e.: beverages, oral rinses, etc.) and eat only soft food if necessary for at least four hours.

Profisil® Fluoride Varnish Plus is to be applied every six months or as recommended by clinician.

Information for the patient

5 % sodium fluoride varnish was applied to your teeth today. This treatment safely delivers a protective coating to your tooth surfaces. To obtain the maximum benefit, we ask that you follow these recommendations after you leave our office:

Do not brush or floss for at least four hours. If possible, wait until tomorrow morning to resume to normal oral hygiene.

Avoid hot drinks and products containing alcohol (i.e.: beverages, oral rinses, etc.) and eat only soft food if necessary for at least four hours.

Cleaning and Disinfection

The components of Profisil® Fluoride Varnish Plus are intended for single use only.

Storage Conditions

Store dry, protected from sunlight.

The product must be stored between 10°C/50°F – 25°C/77°F.

Composition

Polydimethylsiloxanes, 5 % sodium fluoride, corresponding to 22.6 mg / g fluoride ions, hydroxyapatite, calcium salts, phosphate salts, steviol glycosides (stevia)*, film forming organic polymer, flavor*.

*not included in the "unflavored" version

This formula is not made with products containing gluten, peanuts, tree nuts, soy, sesame, corn, shellfish, egg or dairy.

Disposal

The empty blister foil and brushes should be disposed in accordance with the local regulation.

Trademarks

Profisil® is a registered trademark of Kettenbach GmbH & Co. KG.

Note

Selected Kettenbach materials / products are available in certain markets only.

For dental professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of licensed dental professionals.

If serious incidents occur that are related to the product, please immediately contact the manufacturer and the competent authority.

Limitation of Liability

Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the legal basis asserted, including warranty, contract, negligence, or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development.

However, users must comply with and consider all recommendations and information in connection with any use of the product.

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Profisil® es una marca comercial registrada de Kettenbach GmbH & Co. KG.