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Product Description

Visalys® Temp, developed by Kettenbach, is a highly esthetic, tooth-colored composite for fabricating temporary dental restorations that is also suitable for the anterior area. The two-component system is based on a multifunctional acrylic composite and is free of Bisphenol A and its derivatives. It has properties similar to filling materials and is available in the following shades: A1, A2, A3, A3.5, B1, BL. Temporary restorations fabricated with Visalys® Temp can also be supplemented and customized with low-viscosity, light-curing composites.

Indications / Techniques

- Fabrication of temporary crowns, partial crowns, bridges, inlays, onlays, and veneers
- Customization of preformed acrylic and metal crowns
- Fabrication of long-term temporary restorations

Patient Target Group

Patients being treated in the context of a dental procedure.

Warnings

Do not use with known allergies to methacrylic acid ester, tert-butylperoxy-3,5,5-trimethylhexanoate, or 4-methoxy-

phenol. Allergic reactions to the product cannot be ruled out in sensitive persons. Sensitization as a result of skin contact is possible.

Visalys® Temp temporary restorations are intended to be used for temporary restorations and are not to be used as permanent restorations. Residual methacrylate-based material may change the setting behavior of silicone impression taking and bite registration materials. If a restoration has been fabricated beforehand, the preparation and surrounding tissue must be carefully cleaned with gauze soaked in alcohol and water spray before taking an impression. Eugenol-based temporary luting cements may inhibit the curing of the permanent composite cements that may subsequently be used for the final cementation.

Cautions

Do not use after expiration date. Do not leave any residue of the temporary crown and bridge material in the sulcus/on the tooth or oral cavity. Do not swallow material! If swallowed: Consult a medical doctor if problems arise or persist. Avoid contact with the eyes. If there is accidental contact with the eyes: Rinse immediately and thoroughly with an eye wash or water. Consult a medical doctor if problems arise or persist. Do not inhale grinding dust. Use safety glasses and face mask! When taking

impressions of areas with severe undercuts and wide interdental spaces, standard blocking-out techniques must be used.

Avoid contact with clothing, as the material cannot be removed by dry cleaning.

Safety data sheets are available at www.kettenbach.com.

Anatomical Impressions

To fabricate a temporary crown, bridge, inlay, or onlay, all standard impression materials can be used.

An A-silicone such as Silginat® is particularly well suited for anatomic impression taking because it remains dimensionally stable for several weeks and can be stored. When remaking a new temporary restoration, the impression taken with Silginat® can be reused.

Instructions for Use

Preparation

Take an anatomic impression of the dental arch beforehand. A laboratory fabricated, vacuum-formed splint or, in the case of single crowns, a prefabricated metal or plastic crown can be used instead of an impression for forming the shape. The impression can be cut out before use to eliminate any undercuts and to increase the stability of the temporary restoration. For bar connections of spaced abutment teeth, a channel can be

cut into the impression. In the anterior area, denture teeth can be used as placeholders before taking the impression.

Before applying Visalys® Temp, prepared tooth surfaces of compomers, resin-modified glass ionomers or composites must be isolated, using petroleum jelly for example.

For optimal results we recommend wearing gloves made of vinyl or latex.

Dispensing and Mixing

The pastes are mixed and dispensed using Applyfix 6 dispensing guns DS-50 4: 1/10:1 and the corresponding mixing tips. Only use the enclosed Kettenbach mixing tips included.

The material requires a certain amount of time to flow out after the mixing tip has been attached. The material flow stops as soon as pressure is reduced on the dispenser handle. Any material already cured in the mixing tip must not be extruded with force because otherwise the cartridge and the mixing tip may be damaged.

Dispensing and Applying

Application

1. Select the desired shade, insert the cartridge in the dispensing gun, advance the plunger into the cartridge, remove the cap, and attach a new mixing tip (turn 90° clockwise until it clicks into place; Figure 1, page 95).

Before attaching a mixing tip onto a new cartridge for the first time, discharge a small amount of material until both materials extrude uniformly (the original cartridge cap can be replaced after use).

2. Before each use, extrude and discard a little material (Figure 2, page 95).

This point is defined as the start of mixing so start the stopwatch at the same time.

3. Fill the impression, vacuum-formed splint, or prefabricated crown with Visalys® Temp from the base upwards (Figure 3, page 95).

Keep the end of the mixing tip immersed in the material to avoid trapping any air bubbles.

The curing of the material is influenced by a number of different factors (e.g., mouth temperature, impression process, storage temperature, anatomic impression).

4. Insert and cement the filled mold, applying slight pressure, within 40 sec. of starting the mixing into the patient's mouth (Figure 4, page 95).

5. Remove the temporary restoration from the patient's mouth during the elastic phase between 1:30 min. and 2:30 min. after the start of mixing (Figure 5, page 95).

Visalys® Temp reaches an optimal elastic phase for easy removal on average 2 min. after the start of mixing. Use a probe to check the degree of hardness of excess material intraorally to determine whether the material can be removed from the mouth.

If this is not possible, we recommend applying a pea-sized quantity to the back of the hand directly after filling the impression. When the material has cured, the impression can be removed with the temporary restoration.

6. Finish the Visalys® Temp temporary restoration about 4 min. after the start of mixing (Figure 6, page 95).

After use, replace the sealing cap (alternatively, the used mixing tip can be used to seal the cartridge after carrying out hygiene measures). Before using the cartridge again, remove the cartridge cap (or the filled mixing tip), check the outlet of the cartridge to ensure that no cured material is blocking the openings (clear any blocked openings and discharge a small quantity of material until both materials extrude uniformly). Attach new mixing tips and proceed as described above.

Finishing

Completely remove the inhibition layer after the material has fully cured using alcohol, for example.

Once the material has fully cured (4 min. after the start of mixing), the temporary restoration can be finished using cross-cut burs and polishers, for example.

Cleaning/Disinfection

If necessary, Visalys® Temp can be disinfected with standard aqueous disinfectant.

Customization/Recontouring

Using a bonding agent is not necessary when using a filling composite in combination with Visalys® Temp for customization or recontouring. For other products, please follow the instructions for use.

Cementation

Remove any residual isolating material from the prepared tooth before temporary cementation.

Standard temporary luting materials can be used for cementation.

Residual eugenol-based temporary luting cement may negatively affect the curing of composite cements for the subsequent final cementation. If the use of composite cements

is planned, it is recommended to use a eugenol-free temporary luting cement.

After Cementation

Carefully check the sulcus and the surrounding areas and if necessary remove any residual temporary material or cement.

Repair

Roughen the fracture site and create mechanical retention on the adjacent areas.

Clean using Orthoskavident® C or alcohol and then air dry.

Join or add to newly fabricated restorations with Visalys® Temp or a flowable, light-curing filling composite.

Join or add to existing restorations with Visalys® Temp or a combination of a flowable, light-curing filling composite and a bonding agent.

Cure Visalys® Temp or the composite in accordance with the relevant instructions for use.

Finish as usual once the material has cured.

Cleaning and Disinfection of the Applyfix 6

Remove the cartridge.

Remove any uncured paste using a cloth soaked in alcohol.

The Applyfix 6 dispensing gun can be used several times if properly disinfected. Kettenbach recommends exchanging the gun if there are visible traces of use. Disinfection can be carried out using a commercially available dipping disinfectant. Kettenbach recommends the use of glutaraldehyde-based solutions.

Storage Conditions

Store in a dry place, protected against sunlight and at room temperature (18°C/64°F–25°C/77°F). Avoid exposure to extreme temperature fluctuations.

Do not store in a refrigerator.

Constituents

Main components of Visalys® Temp are approx. 30 vol% inorganic fillers in the size range 0.2–20 µm. The polymer base consists of aliphatic dimethacrylates.

Disposal

The empty cartridges/mixing tips and cured material can be disposed in the household rubbish.

Trademarks

Visalys® Temp, Orthoskavident® C and Silginat® are registered trademarks of Kettenbach GmbH & Co. KG.

Note

Not all Kettenbach products are available in all countries.

For dental professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

If serious incidents occur that are related to the product, please immediately contact the manufacturer and the competent authority.

Limitation of Liability

Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.


The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and information in connection with any use.

Updated: August 23, 2022

Technical data

Working and setting times

00:00–00:40 min.	Pouring Visalys® Temp into the impression and placement in the mouth
	Curing in the mouth
01:30–02:30 min.	Removal from the mouth
	Full curing
From 04:00 min.	Finishing the temporary restoration

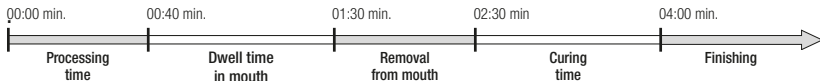Note

Working times apply for products, which have been stored and processed at a temperature of 23°C/74°F and a relative humidity of 50 %. Higher temperatures shorten the working time and lower temperatures extend the working time.

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